




INSTRUCTIONS FOR USE FOR TIMESCO R ENGINES

Timesco Healthcare Ltd Timesco House 3 Carnival Park
Carnival Close Basildon Essex S514 3WN England
T +44 (0) 1268 297 700 F +44 (0) 1268 297 800
E sales@timesco.com timesco.com
E & OE, January 2019. Ref: IL.023/2

HANDLES	ENGINES	
	TIMESCO R (C) 3115.150.50 (2.5v)	TIMESCO R (AA) 3115.150.20 (2.5v)
XLED (Medium C) 3000.500.10	●	
XLED (Small AA) 3000.500.05		●
Xenon (Medium C) 3000.350.10	●	
Xenon (Small AA) 3000.350.05		●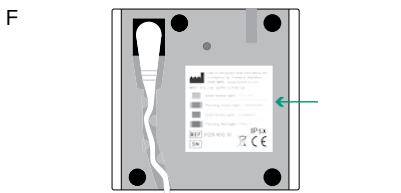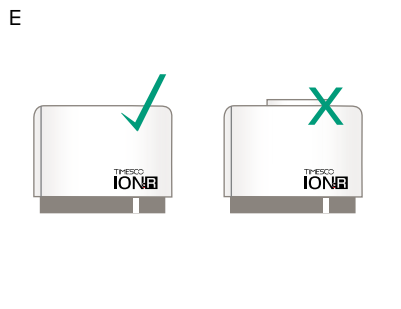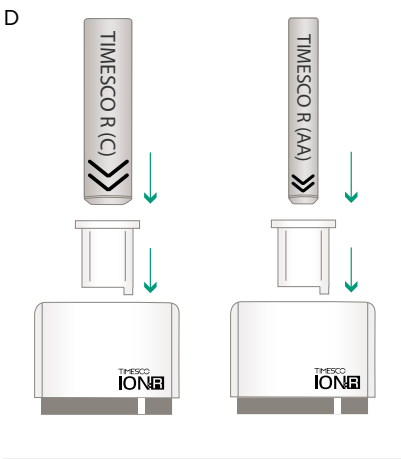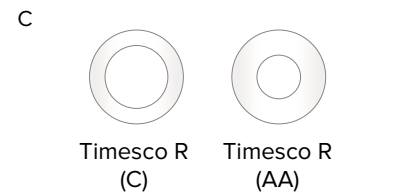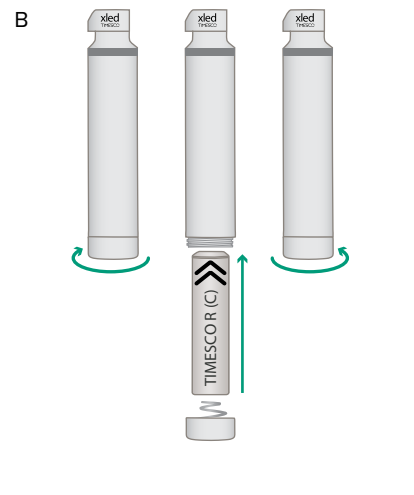

INSTRUCTIONS FOR USE FOR TIMESCO R LITHIUM RECHARGEABLE ENGINES

Timesco R Engines are available in Medium C size and Small AA size. Please select the correct engine for use with the correct handle.

Select the correct engine - Please see compatibility chart - See Figure A.

Inserting the engine into a handle - See Figure B.

Important - Insert the engine into the handle in the correct direction, as shown on the engine. Do not insert the wrong way as this will damage the engine and invalidate the guarantee.

Charging a Timesco R Engine - Choose the appropriate adaptor for the engine - See Figure C. Insert the adaptor and engine into the ION R Charger (3125.100.10) - See Figure D. The adaptor must sit flush with the top of the charger - See Figure E. The light on the front of the charger should turn yellow when the engine is detected. Follow the recharging sequence detailed for the Timesco ION R Recharger until the engine is fully charged - See Figure F. A fully drained TIMESCO R Engine can be fully charged in approximately 1 hour.

Cleaning and Reprocessing
Engines are not autoclavable but may be cleaned (if required) by wiping with an approved disinfectant.

Cautionary
Do not expose to liquids or immerse in liquids.
Do not remove the engine label, as this will invalidate the guarantee.
Do not drop.
Do not expose to extreme temperatures (normal operating range of 0°C to +40°C)
Do not attempt to charge in any other charger than a Timesco ION R Charger.
Do not dispose of in normal waste due to internal batteries and components. Use appropriate disposal methods or contact Timesco for advice.
Do not insert the wrong way as this will damage the engine and invalidate the guarantee.

Engine Specification
TIMESCO R Medium C Size Engine:
• Guaranteed for 7 years or a minimum of 1,000 recharge cycles.
• Fully charged in a Timesco ION R Charger in approximately 1 hour.
• Requires one C size adaptor (3125.100.17) for use with an ION R Charger.

• Operating temperature between 0°C and +40°C.
TIMESCO R Small AA Size Engine:
• Guaranteed for 7 years or a minimum of 1,000 recharge cycles.
• Fully charged in a Timesco ION R Charger in approximately 1 hour.
• Requires one AA size adaptor (3125.100.16) for use with an ION R Charger.
• Operating temperature between 0°C and +40°C.

Part Numbers and Descriptions:

3115.150.20	TIMESCO R Lithium Ion Battery Engine Small AA Size 2.5V
3115.150.50	TIMESCO R Lithium Ion Battery Engine Medium C Size 2.5V
3125.100.10	ION-R Charger without PSU
3125.100.20	ION-R PSU only (including Intl adapters)
3125.100.50	ION-R Charger and PSU Set
3125.100.12	ION-R Charger Left blanking plate only
3125.100.14	ION-R Charger Right blanking plate only
3125.100.16	ION-R Charger adaptor for Small AA Timesco R Engines
3125.100.17	ION-R Charger adaptor for Medium C Timesco R Engines

Rx ONLY Federal (U.S.A) law restricts this device to sale by or on order of a physician.

- Consult instructions for use
- Latex Free
- Do not dispose of in fire
- Not to be disposed of with household waste
- Static sensitive contacts
- Operating temperature

IP5X Limited protection against dust ingress, no protection against liquid ingress

ION, ION R, SHAKERSCOPE, SINGLE USE SKIN and the ION R Recharger are part of the Energy Efficient Systems (EES) range. Designed and manufactured in the UK by Timesco Healthcare Ltd. SHAKERSCOPE is a registered trademark of SHAKERSCOPE Ltd. ION, ION R and SINGLE USE SKIN are trademarks of Timesco Healthcare Ltd.

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