
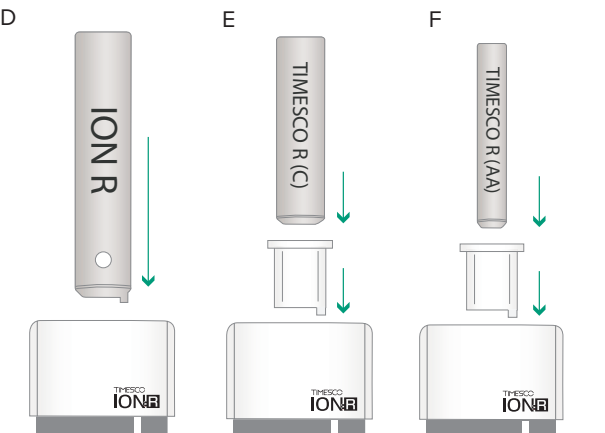
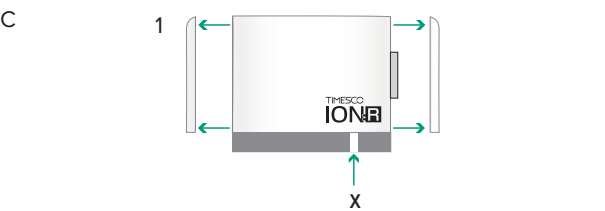
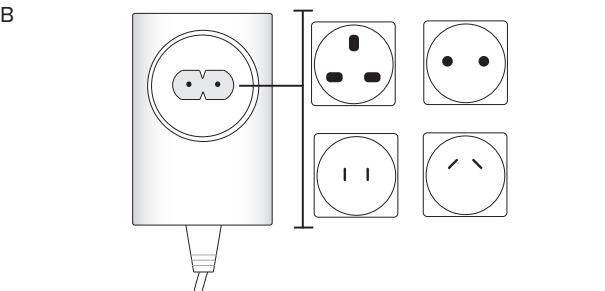
ION R Charger Instructions for Use

Timesco Healthcare Ltd Timesco House 3 Carnival Park Carnival Close
Basildon Essex SS14 3WN England
T +44 (0) 1268 297 700 | F +44 (0) 1268 297 800 | E sales@timesco.com | timesco.com
E & OE, Jan 2018. Ref. IL.022/2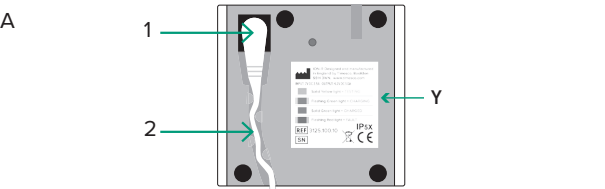

GB ION R CHARGER

Timesco ION R Chargers are to recharge ION R Engines and Timesco R Engines only.

Power supply unit (PSU)

ION R Chargers are not supplied with a PSU, these are available from Timesco separately.

Connecting the PSU - See Figure A

Please note: Up to 3 chargers can be connected together (See Figure C-2) and powered by 1 PSU connected to any charger in the chain.

Select the correct plug for your region and connect it to the PSU – See Figure B

If the PSU is properly connected, the LED (X) should glow white.

Connecting multiple chargers

ION R Chargers can be used as a single port charger or a bank of 2 or 3. To connect the chargers together - See Figure C

Note: Keep the end panels safe for future use.

Charging ION R Engines

The ION R engine only requires charging when the engine advises it is low on charge. See ION R instructions for more information.

Switch the engine off and place in the charger – See Figure D. The charger LED should turn yellow to indicate that the engine is properly inserted. A fully drained engine can be charged in approximately 1 hour.

Charging Timesco R Engines

Timesco R Engines are available in AA and C sizes. Ensure the appropriate adaptor is correctly inserted into the charging port before inserting an engine. Align the engine to fit correctly in the adaptor and charge port (see figure E & F). Charger LED should turn yellow. A fully drained engine can be charged in approximately 1 hour.

LED Operating Sequence (Y)

The LED on the front of the charger indicates the following:

- White – Power but no engine inserted
- Yellow – Charger checking state of charge or health of engine.
- Green Flashing – Charging
- Green Solid – Fully Charged
- Red Flashing – Engine fault, replace engine or contact Timesco or your representative/distributor.

Guarantee and General Maintenance:

ION R Charger: Guaranteed for 10 years

PSU: Guaranteed for 5 years

Timesco ION R Chargers will use very little power when there is no engine inserted or the engine is fully charged.

Cleaning and Maintenance

Clean external surfaces with an approved disinfectant wipe and follow manufacturer's instructions.

Do not use spray disinfectants.

Do not attempt to clean inside the charging port or touch the charging pins.

Do not machine clean, sterilise or immerse in liquids

Cautions:

Do not allow any foreign objects into the charger port.

Do not drop.

Do not expose to excessive temperatures (normal operating range of 0°C to +40°C)

Do not attempt to charge any other batteries or devices in the ION R Charger that are not approved by Timesco.

Do not dispose of in normal waste due to internal components.

Use appropriate disposal methods or contact Timesco for advice.

If radio frequency interference with monitoring equipment should occur, connect the recharger to a different power source

Do not touch the charger terminals.

Do not short circuit.

Rx ONLY

Federal (U.S.A) law restricts this device to sale by or on order of a physician.

i

Consult instructions for use

Latex Free

Do not dispose of in fire

Not to be disposed of with household waste

Static sensitive contacts

IP5X

Limited protection against dust ingress, no protection against liquid ingress

3125.100.10	ION-R Charger without PSU
3125.100.20	ION-R PSU only (including Intl adapters)
3125.100.50	ION-R Charger and PSU Set
3125.100.12	ION-R Charger Left blanking plate only
3125.100.14	ION-R Charger Right blanking plate only
3100.300.55	ION-R Lithium Ion Rechargeable Engine
3115.150.20	Timesco R Lithium Ion Battery Engine - Small AA
3115.150.50	Timesco R Lithium Ion Battery Engine - Medium C
3125.100.16	ION-R Charger Adaptor for Timesco R Engine Small AA
3125.100.17	ION-R Charger Adaptor for Timesco R Engine Medium C

The EES range is protected by the following granted patents:
GB2451650, GB2486858 and pending patents 08788266.8, 12/671938, 06744349.9 and 114952.3.
Designed and manufactured in the UK by Timesco Healthcare Ltd.
ION, ION R, SINGLE USE SKIN and TIMESCO are trademarks of Timesco Healthcare Ltd.

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