

TIMESCO™

Wira-Flexi



SWEDISH DESIGN 



A UNIVERSAL MEDICAL TUBE, DRAIN & CATHETER FIXATION DEVICE

Wira-Flexi isn't a one-size-fits-all solution. We understand different catheters and tubes require specific levels of securement. That's why Wira-Flexi comes in two convenient sizes: Large & Extra Large.

LARGE (90MM X 35MM) C80.805

IDEAL FOR MOST CATHETERS & TUBES WITH A DIAMETER OF UP TO 20MM, INCLUDING:

- Epidural catheters
- Indwelling needles
- Peripheral intravenous lines
- Urinary catheters
- Chest drains

EXTRA LARGE (45MM X 150MM) C80.810

PROVIDES SUPPORT FOR LARGER TUBES & CATHETERS EXCEEDING 20MM, INCLUDING:

- Central venous catheters
- Gastrostomy tubes
- Chest drains (large)
- Wound drains
- Nephrostomy tubes

No matter the size or type of tubing or catheter, Wira-Flexi offers the same level of comfort, security, and ease of use.

The breathable material conforms to body contours for optimal comfort, while the strong, resealable velcro keeps everything securely in place.




3 WAYS TO SECURE

SECURELY HOLD TUBES, DRAINS
& CATHETERS IN PLACE

HYPO-ALLERGENIC & LATEX FREE ADHESIVE

PLACE THE DEVICE DIRECTLY OVER TUBING, DRAINS AND CATHETERS SECURELY, UTILISING THE ADHESIVE ON THE UNDERSIDE OF THE DEVICE, WHILE REMAINING GENTLE ON THE SKIN TO MINIMISE IRRITATION

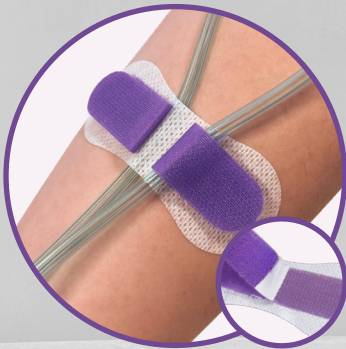

STRONG, RE-SEALABLE VELCRO

FIX MULTIPLE TUBES IN PLACE USING THE SOFT, FLEXIBLE VELCRO TABS. THE VELCRO IS STRONG AND RESEALABLE SUITABLE FOR MULTIPLE ADJUSTMENTS AND TUBING CHANGES.



INTERNAL ADHESIVE FOR SAFE SNAG SPACE

USE THE DISCREET INTERNAL ADHESIVE PADS FOR EXTRA SECURITY WHEN CREATING 'SLACK' OR 'SNAG SPACE' FOR TUBE MOVEMENT. THIS IS USEFUL FOR THOSE REQUIRING HIGH LEVEL CARE OR THOSE WANTING SECURITY AND MOBILITY



Easy to use, even with one hand, Wira-Flexi is hypo-allergenic and latex-free for minimised skin irritation. With a long lasting wear time, reducing dressing changes for both patients and healthcare professionals.

HOW TO APPLY

SAFE AND FLEXIBLE FIXATION FOR CATHETER, TUBING AND DRAINAGE.

1

CLEAN THE SKIN WHERE THE FIXATION IS TO BE ATTACHED. THE SKIN MUST BE DRY WHEN FIXING.

2

OPEN THE OUTER PACKAGING BY SPLITTING THE TABS ON THE SIDE, REMOVE THE WIRA-FLEXI DEVICE.

3

REMOVE THE BACKING & APPLY THE DEVICE TO THE SKIN.






4

OPEN THE VELCRO ON THE UPPER SIDE OF THE STRIP & PLACE THE TUBE/CATHETER IN ONE OF THE 2 VELCRO HOLDERS, CLOSE THE VELCRO TO SEAL.

THE VELCRO CAN BE OPENED & CLOSED TO ADJUST THE TUBING/CATHETER.

The Wira-Flexi fixation device utilises a velcro system that allows for easy application and removal of the tube or catheter. You can open and close the velcro multiple times to secure the tubing in place.

When it's time to replace the tube or catheter, simply open the velcro and attach the new one. The device can remain in place during tube changes, minimising discomfort and hassle.

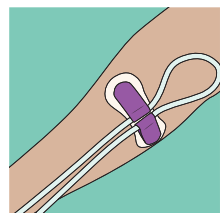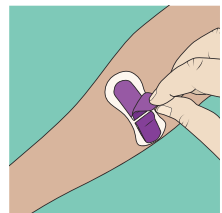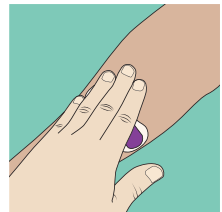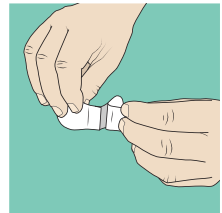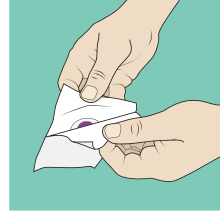

HOW TO REMOVE

1

REMOVE THE TUBING OR CATHETER FROM THE DEVICE BY OPENING THE VELCRO.

2

HOLD THE SURROUNDING SKIN TAUGHT

3

PEEL THE DEVICE IN THE DIRECTION OF ANY HAIR FOLLICLES

4

DISPOSE OF THE DEVICE IN ACCORDANCE WITH YOUR WASTE MANAGEMENT GUIDELINES.

1

2

3

4

DIVERSE APPLICATIONS



Wira-Flexi goes beyond a one-size-fits-all solution. It's a versatile champion for secure and comfortable tube fixation across various medical scenarios. From the critical care setting where it secures central venous catheters and chest drains, to home care where it gently holds feeding tubes and urinary catheters, Wira-Flexi adapts to a wide range of applications.

Anesthesia, dialysis, wound drainage, epidural catheters, and neurological monitoring – the list goes on! This adaptability ensures patient comfort and securement during a multitude of medical procedures.

POPULAR USES



FEEDING CATHETER



PICC LINE CATHETERS



TRANSFUSION THERAPY



LONG TERM NEUROLOGY (EKG)



UROLOGY



DIALYSIS



ANAESTHESIA



GASTROENTEROLOGY



MATERNITY AND DELIVERY



EXPERIENCE THE WIRA-FLEXI DIFFERENCE

Busy medical professionals deserve a helping hand. Wira-Flexi provides secure fixation for catheters and tubes, reducing disruptions and allowing you to focus on what matters most.

EXPERIENCE:

- Comfortable, breathable material
 - Secure, resealable velcro hold
 - Easy one-handed application
 - 5-day wear time for fewer dressing changes
-
-



HOW DOES WIRA-FLEXI WORK FOR YOU?

HOME CARE

- Maintain patient independence with a secure yet low-profile design.
- Personable, allows users or carers to adapt to their needs and preferences.



HOSPITAL CARE

- Reduce accidental dislodgements and potential complications.
- Strong velcro ensures catheters stay securely in place.



COMMUNITY CARE

- Flexible design conforms to body contours for optimal comfort.
- Allows for freedom of movement without compromising fixation.



HIGH DEPENDENCY CARE

- Minimises the risk of snagging on clothing or bedding.
- Provides reliable but gentle hold for even the most delicate of patients.



[TIMESCO.COM/WIRA-FLEXI](https://www.timesco.com/wira-flexi) / +44 (0)1268 297 710 / SALES@TIMESCO.COM